


## BD Vacutainer® Plus Plastic Citrate Tube Draw Volume Guide

Sufficient volume achieved if blood drawn falls above minimum fill indicator. For blood transfer, **do not** fill above illustrated dashed maximum line.

**Note:** The quantity of blood drawn into evacuated tubes varies with altitude, ambient temperature, barometric pressure, tube age, venous pressure and filling technique.


Ref. # 363083

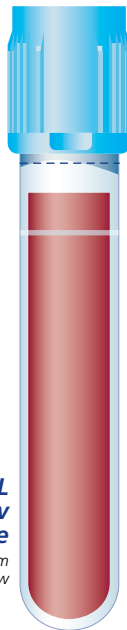

**2.7 mL  
Draw  
Tube**

13 mm x 75 mm  
Full Draw

Ref. # 363080

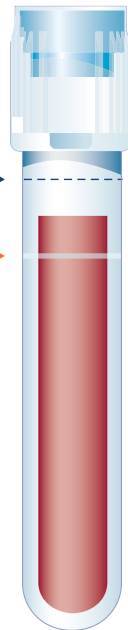

**1.8 mL  
Draw  
Tube**

13 mm x 75 mm  
Full Draw

Maximum Fill\*

Minimum Fill  
Indicator

**Now with  
Minimum Fill  
Indicator**

Represents minimum  
volume of blood  
required for  
appropriate analysis

\*According to CLSI guideline,  
Dec. 2003, Doc. H1-A5, Vol. 23, No. 33.